



**A Brand Division of American Tooth Industries**

1200 Stellar Drive | Oxnard, CA 93033-2404

(805) 487-9868 | FAX: (805) 483-8482

info@americantooth.com | americantooth.com

## Instruction Sheet

### Justi High-Impact Denture Base PMMA Disc

Justi High-Impact Denture Base PMMA Discs are intended for use in milling full and partial denture prosthetic appliances. Justi Discs are available in 98.5mm diameter to fit most open type, 5-axis milling machines. Discs are available in the heights; 20mm, 25mm, and 30mm, and are available in four shades; light pink, light red pink, dark pink, and regular original.

#### Instructions:

1. Mount the disc in the milling machine jig following the equipment manufacturer's instructions.
2. After milling, remove the disc from the machine and remove the sprues. Minimal finishing of the denture base may be necessary. Follow your laboratory's recommended technique for finishing, festooning, and polishing dentures.
3. In order to finish the denture pre-manufactured teeth, need to be bonded to the denture base. American Tooth Industries recommended Justi Imperial ViVO teeth.

#### Important Considerations:

- Avoid heat and light during storage
- Single use only
- Use care and judgement to avoid generating high heat during milling and polishing, as it may damage the denture base
- Do not use autoclave to sterilize

**Precautions:** The following safety and warning information describes how to use the Justi High-Impact Denture Base Discs in a safe and risk-free manner.

- Keep product dust away from eyes
- Do not inhale dust particles during grinding.
- Use local vacuum suction and suitable mouth/face protection during manual machining at the workplace
- If ingested contact regional Poison Control Center
- Comply with pertinent SDS (safety data sheets)

American Tooth Industries (ATI) has tested and pre-qualified a list of open milling machine systems. ATI only recommends use of these systems. ATI shall not have any liability to the dental laboratory or any other person or entity for any indirect, incidental, special or consequential damages whatsoever, including but not limited to, los of revenue or profit, lost or damaged data or other commercial or economic loss, even if ATI has been advised of the possibility of such damages., or they are foreseeable. Also, ATI is not responsible for and shall not have obligation or liability for claims by third party.